

Install the IMROC-IR Control Console

Steps:

1. Place the Control Console in the Control Room


Bring the IMROC-IR Console to your MRI Control Room and place it on a table for ready access during a scan by the MRI Technician. For the exact space requirements, click [here](#).

The Console requires connection to a standard electrical outlet. Typically, it should be located within 15 m of the EOU to enable attachment of the data cable that is supplied with the System.

In addition, leave space on the table for placement of the external stereo speakers (supplied).

2. Connect the Control Console

1. Attach RS-232 female connector to its port, and use a small screwdriver to securely tighten its screws.
2. Insert the 12V electrical power supply plug in its connector.
3. Insert the external speaker stereo mini-plug in its connector.

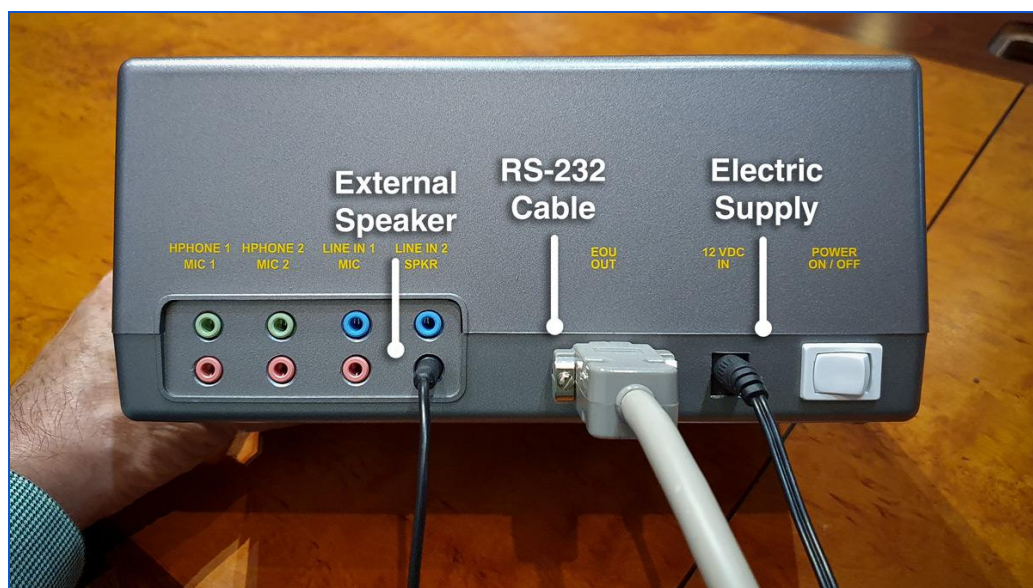

3. Connect the Control Console to Electric Power

Insert the plug of the 12V power supply into a standard electrical power receptacle.

Do *not* power on the Console yet.

You have completed the basic IMROC-IR Console installation.