

COVID-19 CLEANING NOTICE

For FOMRI™ and OptoACTIVE™ Products

Especially during the COVID-19 pandemic, we remind you that maintaining maximum personal hygiene for Optoacoustics products requires consistent cleaning, along with replacement of all single-use sanitary components after each use.

1. Wear personal protective equipment (PPE) while cleaning.
2. Use only hospital-standard cleaning alcohol.
3. Use only hospital-standard cleaning pads.
4. Pads must be moist, not soaked – do not let alcohol drip into sensitive components.
5. Be careful not to overbend or apply excess pressure to the various cable connections.


FOMRI MICROPHONE INSTRUCTIONS

Gently wipe the following microphone parts with a sterile cleaning pad moistened with alcohol:

- Connector box
- Flexible gooseneck
- Microphone housing

Do not let alcohol drip into the microphone housing.

Replace the hygienic patient pop screen.

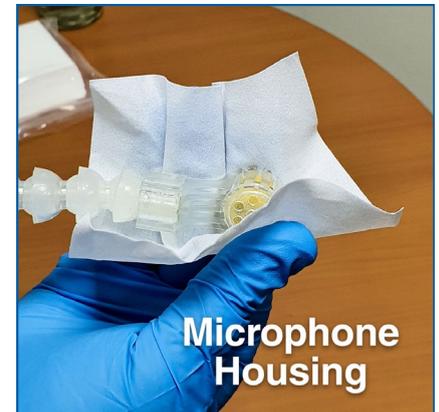

OptoACTIVE HEADSET INSTRUCTIONS

Gently wipe the following headset parts with a sterile cleaning pad moistened with alcohol:

- Backside of each earpiece
- Cushion padding of each earpiece

Do **not** clean the inside mesh fabric that covers internal headphone speakers.

Replace the hygienic patient earpad covers.



PHONE +1 866-867-5029 US Toll Free
+972 3-634-4488
FAX +972 3-634-9292
EMAIL info@optoacoustics.com

Optoacoustics Ltd, 17 Hanotea Street 73160 Mazor Israel